geemarc™
Hear the Difference

CL7400BT

En	Amplified Bluetooth Digital Headset with Quality Sound
Fr	Casque Bluetooth Amplifié avec Qualité Hifi
D	Verstärkter Bluetooth Kopfhörer mit kristallklare Klangqualität
Español	Auricular con amplificación digital vía Bluetooth con sonido de calidad
Italiano	Cuffia digitale amplificata Bluetooth con suono di qualità
Nederlands	Versterkte Bluetooth digitale koptelefoon met kwaliteitsgeluid
עברית	אוזניות בלוטות' דיגיטליות עם הגברה בעלות צליל איכותי
Svenska	Förstärkt Bluetooth Digital Headset med ljudkvalitet
Magyar	Erősítős digitális Bluetooth headset minőségi hanggal
Dansk	Forstærket Bluetooth digital hovedtelefon med høj lyd kvalitet
Norsk	Forsterkende bluetooth digital hodetelefoner med høy lyd kvalitet
PolSKI	Cyfrowa słuchawka Bluetooth ze wzmocnieniem, o doskonałej jakości dźwięku

En	Unpacking	It	Estrarre dall'imballaggio	Magyar	Kicsomagolás
Fr	Contenu du Colis	NL	Uitpakken	Dansk	Udpakning
De	Auspacken	עברית	פריקה	Norsk	Innholdet i pakken
Esp	Desempaques	Svenska	Packa upp	Polski	Zawartość

a

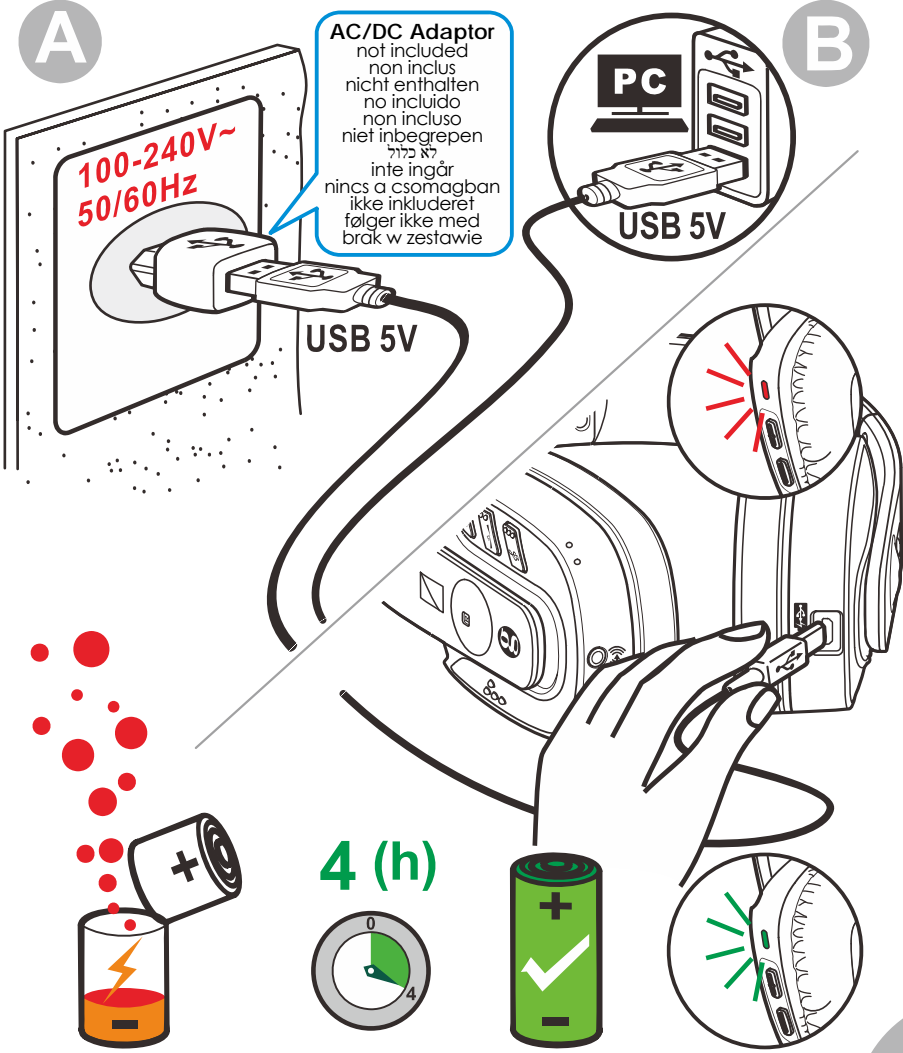
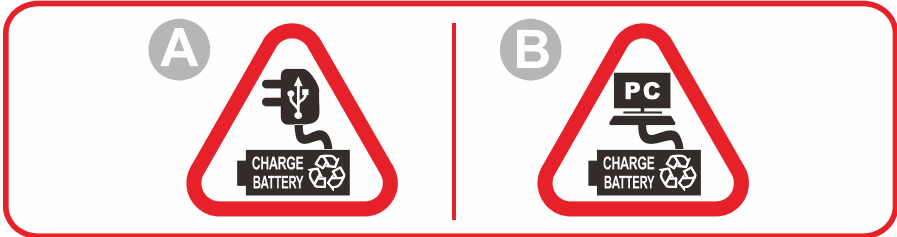
Magyar Charging
 Dansk Chargement
 Norsk Aufladen
 Polski Carga

It Ricarica
 NL Opladen
 עברית טעינה
 Svenska Laddar

En Töltés
 Fr Opladning
 De Lading
 Esp Ładowanie

b

En	Pairing	It	Accoppiare	Magyar	Csatlakoztatás
Fr	Enregistrement	NL	Paarvorming	Dansk	Sammenkobling
De	Anmelden	עברית	התאמה	Norsk	Paring
Esp	Emparejamiento	Svenska	Parning	Polski	Parowanie

C

Mobile phone / TV / PC /

1

ON

3" (s)

2

ON

3

4

CL7400BT
P/W/PIN? **0000**
OK Cancel

5

OK

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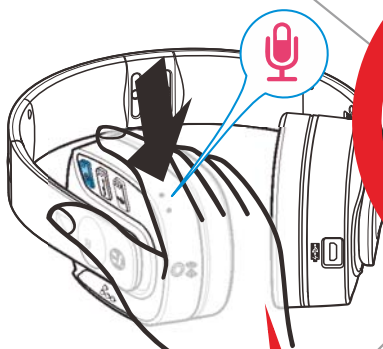
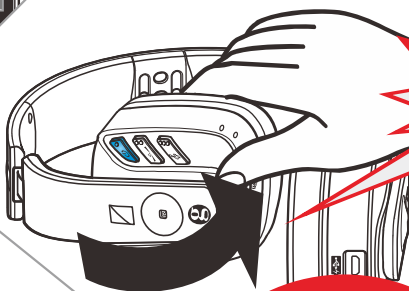
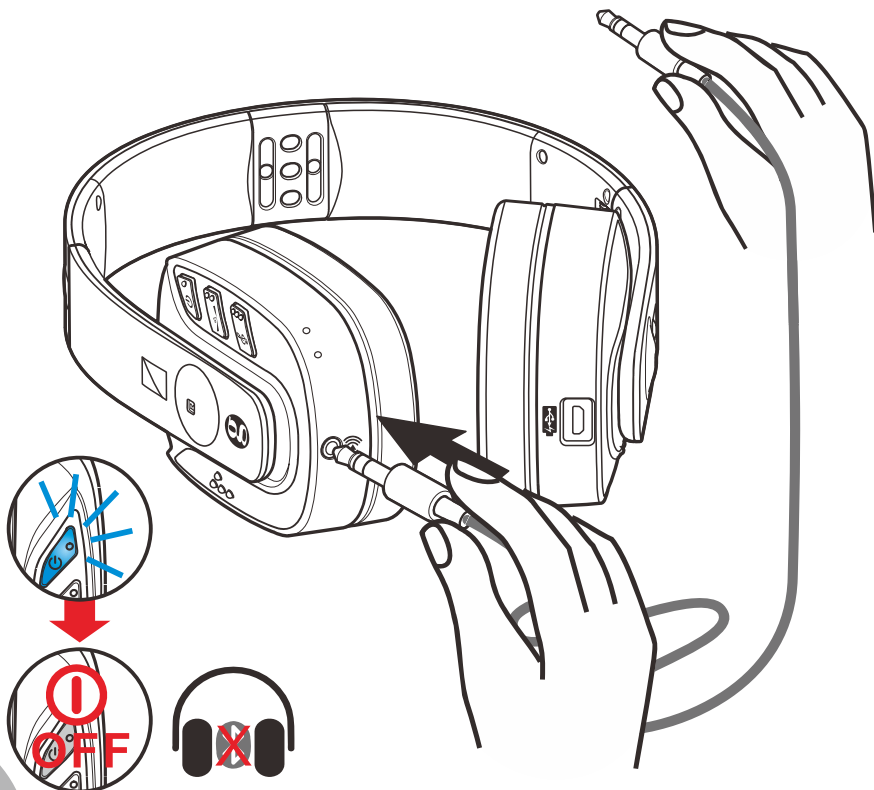
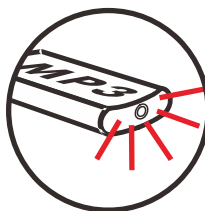
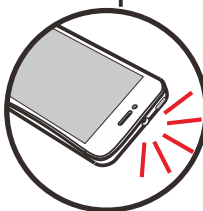
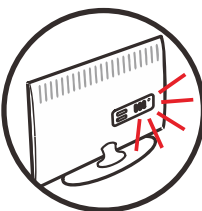
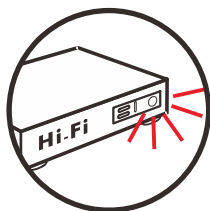
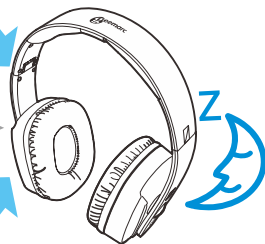
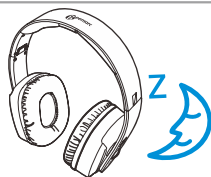
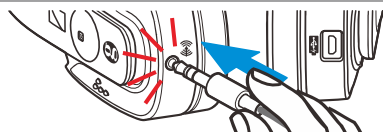
60" (s)

Safety & Care advice

To make the best use of this product and to use it safely, please read these instructions thoroughly before use and keep safe for future reference.

Warning: This product can be very loud. Take care to prevent others from using this product, if they do not have the same level of hearing loss. Excessive sound pressure from earphones and headphones can cause hearing loss.

Caution: Electric shock. DO NOT insert unattached plug heads in to mains power sockets. Plug heads **MUST** be fitted to the power adaptor.

The unit does not incorporate an integral power on/off switch. To disconnect the power, either switch off supply at the mains power socket or unplug the AC adaptor. When installing the unit, ensure that the mains power socket is readily accessible.

Caution: Risk of explosion if battery is replaced by an incorrect type. Dispose of used batteries according to the instructions.

DO NOT cover the ventilation.

DO NOT remove the ear pad as it is very difficult to replace.

Keep the unit away from heat sources.

DO NOT expose the unit to direct sunlight.

Clean the Headset and ear pads regularly.

DO NOT modify the unit in any way.

Please note: The unit may be subject to interference from appliances that generate strong electrical or magnetic fields e.g. microwaves, mobile phones, lighting transformers etc.

Battery Warnings:

- **DO NOT** attempt to dismantle.
- **DO NOT** dispose of in fire.
- **DO NOT** swallow.
- Keep away from children.
- **DO NOT** short-circuit the supply terminals.
- **DO NOT OPEN**, no user replaceable part inside, refer servicing to qualified personnel.
- When the headphones are not in use or the batteries are charging, switch **OFF** the headphones to avoid damaging the batteries.
- The batteries must be fully charged when use for the first time. An initial 4 hours charge is required.

Troubleshooting:

• **NO SOUND**

1. Ensure the USB cable is properly connected to the power adapter(not including) and the charging connector on the headphone.
2. Ensure that the headphone On/Off switch is switched to the On Position and the switch is illuminant Blue.
3. The headphones battery charge level may be too low. Please connect the USB cable to recharge the batteries until the green light is illuminated on the left hand headset.
4. Ensure that your mobile phone, DVD, TV set, Hi-Fi system or audio component is switched on and set Bluetooth ON and paired with the headphone.
5. Use the headphones volume control to increase the sound level (only when use as a wireless headphone).
6. The audio signal input level is too low. Increase the volume from the audio source of mobile phone, Hi-Fi, CD, or TV, etc.
7. The paired audio/video equipment may not be in playback mode. Start playback on by the equipment.
8. The headphones output level is set too low. Adjust the volume to a suitable level.

• **DISTORTION**

9. Restart the Pairing process (see Page 3).
- 10.The headphone's batteries charge level may be too low. Recharge the batteries.
- 11.Ensure that the volume setting is correct.
- 12.The headphones may be too far away from the music player. Move closer to it.
- 13.The audio signal input level is too strong. decrease the volume from the audio source of mobile phone, Hi-Fi, CD, or TV, etc.

Customer support

For product support and help, visit our website at www.geemarc.com

For our Customer Helpline

Telephone: **01707 384438** Fax: 01707 832529

General information

Features

- Headset Digital technology Bluetooth V4.0.
- High quality stereo sound.
- Digital Pairing System.
- Electronic Volume Control.
- Digital Balance/Tone adjustment.
- Rechargeable by micro USB socket.
- Jack 3.5mm Line in feature for wired headphone application.
- Operating distance up to 10 metres in open field.
- Hands-free can be used via Bluetooth.

Technical specifications

Operating voltage	: 3.7V 1000mA (built-in Lithium rechargeable battery)
Charging voltage	: USB 5V
Frequency response	: 20 Hz – 20 KHz (electrical)
Distortion	: 0.5 %
Signal-to-noise ratio	: 75 dB
Channel separation	: 60 dB
Range	: 10 meters (open area)

Guarantee

From the moment your Geemarc product is purchased, Geemarc guarantee it for the period of two years. During this time, all repairs or replacements (at our discretion) are free of charge. Should you experience a problem then contact our Helpline or visit our website at www.geemarc.com.

The guarantee does not cover accidents, negligence or breakage to any parts.

The product must not be tampered with or taken apart by anyone who is not an authorised Geemarc representative.

The Geemarc guarantee in no way limits your legal rights.

The product is covered by the legal guarantee of conformity as provided by applicable law.

Important: YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM.

Please note: The guarantee applies to the United Kingdom only.

EEC Declaration

Geemarc Telecom SA hereby declares that this product is in compliance with the essential requirements and other relevant provisions of the Radio and Telecommunications Terminal Equipment Directive 1999/5 EEC.

The declaration of conformity may be consulted at www.geemarc.com

To prevent possible hearing damage, DO NOT listen at high volume levels for long periods.

This device complies with Industry Canada Licence-exempt RSSs. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference, including interference that may cause undesired operation of the device.

FCC Statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Warning: Changes or modifications to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Re-orient or relocate the receiving antenna.
 - Increase the separation between the equipment and receiver.
 - Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
 - Consult the dealer or an experienced radio TV technician for help.
-

Disposal information

Disposal of old electrical & electronic equipment (applicable in the European Union and other European countries with separate collection systems)

The symbol on the product or its packaging indicates that this product may not be treated as household waste. Instead it should be handed over to the applicable collection point for the recycling of electrical and electronic equipment.

By ensuring this product is disposed of correctly, you will help prevent potential negative consequences for the environment and human health, which could otherwise be caused by inappropriate waste handling of this product.

For more detailed information about recycling this product, please contact your local council office, your household waste disposal service or the shop where the product was purchased.

All electrical and electronic products including batteries should be disposed of separately from the municipal waste stream via designated collection facilities appointed by the government or the local authorities.

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