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Warning: Please read this instruction manual carefully before using the instrument and keep it for future use.


A. SAFETY INSTRUCTIONS

- _ The device is intended exclusively for private and not for commercial use. Only use this device as describe in this instruction manual.
- _ Do not place any part of this appliance in water or any other liquid.
- _ Keep it away from heat sources such as radiators, to avoid deformation of plastic parts.
- _ This device is intended for adult use in homes only.
- _ The device is not suitable for use on neonatal patients, pregnant women, patients with implanted, electronical devices, patients with pre-eclampsia, premature ventricular beats, atrial fibrillation, peripheral, arterial disease and patients undergoing intravascular therapy or arterio-venous shunt or people who received a mastectomy. Please consult your doctor prior to using the unit if you suffer from illnesses.
- _ The device is not suitable for measuring the blood pressure of children. Ask your doctor before using it on older children.
- _ The device is not intended for patient transport outside a healthcare facility.
- _ This device is intended for no-invasive measuring and monitoring of arterial blood pressure. It is not intended for use on extremities other than the wrist or for functions other than obtaining a blood pressure measurement.
- _ Do not confuse self-monitoring with self-diagnosis. This unit allows you to monitor your blood pressure. Do not begin or end medical treatment without asking a physician for treatment advice.
- _ If you are taking medication, consult your physician to determine the most appropriate time to measure your blood pressure. Never change a prescribed medication without consulting your physician.
- _ Do not take any therapeutic measures on the basis of a self-measurement. Never alter the dose of a medicine prescribed by a doctor. Consult your doctor if you have any question about your blood pressure.
- _ When the device was used to measure patients who have common arrhythmias such as atrial or ventricular premature beats or atrial fibrillation, the best result may occur with deviation. Please consult your physician about the result.
- _ When using this device, please pay attention to the following situation which may interrupt blood flow and influence blood circulation of the patient, thus cause harmful injury to the patient: connection tubing kinking too frequent and consecutive multiple measurements; the application of the cuff and its pressurization on any wrist where intravascular access or therapy, or an arterio-venous (A-V) shunt, is present; inflating the cuff on the side of a mastectomy.
- _ Warning: Do not apply the cuff over a wound; otherwise it can cause further injury.
- _ Do not inflate the cuff on the same limb which other monitoring ME equipment is applied around simultaneously, because this could cause temporary loss of function of those simultaneously-used monitoring ME equipment.
- _ On the rare occasion of a fault causing the cuff to remain fully inflated during measurement, open the cuff immediately. Prolonged high pressure (cuff pressure 300mmHg or constant pressure \geq 15mmHg for more than 3 minutes) applied to the wrist may lead to an ecchymosis.
- _ Please check that operation of the device does not result in prolonged impairment of patient blood circulation.
- _ When measurement, please avoid compression or restriction of the connection tubing.
- _ The device cannot be used with HF surgical equipment at the same time.
- _ This device is contraindicated for any female who may be suspected of, or is pregnant. Besides providing inaccurate readings, the effects of this device on the fetus are unknown.
- _ Too frequent and consecutive measurements could cause disturbances in blood circulation and injuries.
- _ This unit is not suitable for continuous monitoring during medical emergencies or operations. Otherwise, the patient's wrist and fingers will become anaesthetic, swollen and even purple due to a lack of blood.
- _ When not in use, store the device in a dry room and protect it against extreme moisture, heat, lint, dust and direct sunlight. Never place any heavy objects on the storage case.


- _ The equipment is not AP/APG equipment and not suitable for use in the presence of a flammable anesthetic mixture with air or with oxygen or nitrous oxide.
- _ Warning: No servicing/maintenance while the ME equipment is in use.
- _ To avoid measurement errors, please avoid the condition of strong electromagnetic field radiated interference signal or electrical fast transient/burst signal.
- _ The blood pressure monitor and the cuff are suitable for use within the patient environment. If you are allergic to polyester, nylon or plastic, please don't use this device.
- _ If you experience discomfort during a measurement, such as pain in the wrist or other complaints, press the START/STOP button to release the air immediately from the cuff. Loosen the cuff and remove it from your wrist.
- _ If the cuff pressure reaches 40 kPa (300 mmHg), the unit will automatically deflate. Should the cuff not deflate when pressures reaches 40 kPa (300 mmHg), detach the cuff from the wrist and press the START/STOP button to stop inflation.
- _ Before use, make sure the device functions safely and is in proper working condition. Check the device, do not use the device if it is damaged in any way. The continuous use of a damaged unit may cause injury, improper results, or serious danger.
- _ Do not wash the cuff in a washing machine or dishwasher!
- _ The service life of the cuff may vary by the frequency of washing, skin condition, and storage state. The typical service life is 10000 times.
- _ The operator shall not touch output of batteries and the patient simultaneously.
- _ Cleaning :Dust environment may affect the performance of the unit. Please use the soft cloth to clean the whole unit before and after use. Don't use any abrasive or volatile cleaners.
- _ Keep the unit out of reach of infants, young children or pets to avoid inhalation or swallowing of small parts. It is dangerous or even fatal.
- _ Be careful to strangulation due to cables and hoses, particularly due to excessive length.
- _ At least 30 min required for ME equipment to warm from the minimum storage temperature between uses until it is ready for intended use. At least 30 min required for ME equipment to cool from the maximum storage temperature between uses until it is ready for intended use.
- _ Wireless communications equipment such as wireless home network devices, mobile phones, cordless telephones and their base stations, walkie-talkies can affect this equipment and should be kept at least a distance d away from the equipment. The distance d is calculated by the MANUFACTURER from the 80 MHz to 5.8 GHz column of Table 4 and Table 9 of IEC 60601-1-2:2014, as appropriate.
- _ There is no luer lock connectors are used in the construction of tubing, there is a possibility that they might be inadvertently connected to intravascular fluid systems, allowing air to be pumped into a blood vessel.
- _ Please use the device under the environment which was provided in the user manual. Otherwise, the performance and lifetime of the device will be impacted and reduced.

B. INTRODUCTION

DISPLAY READINGS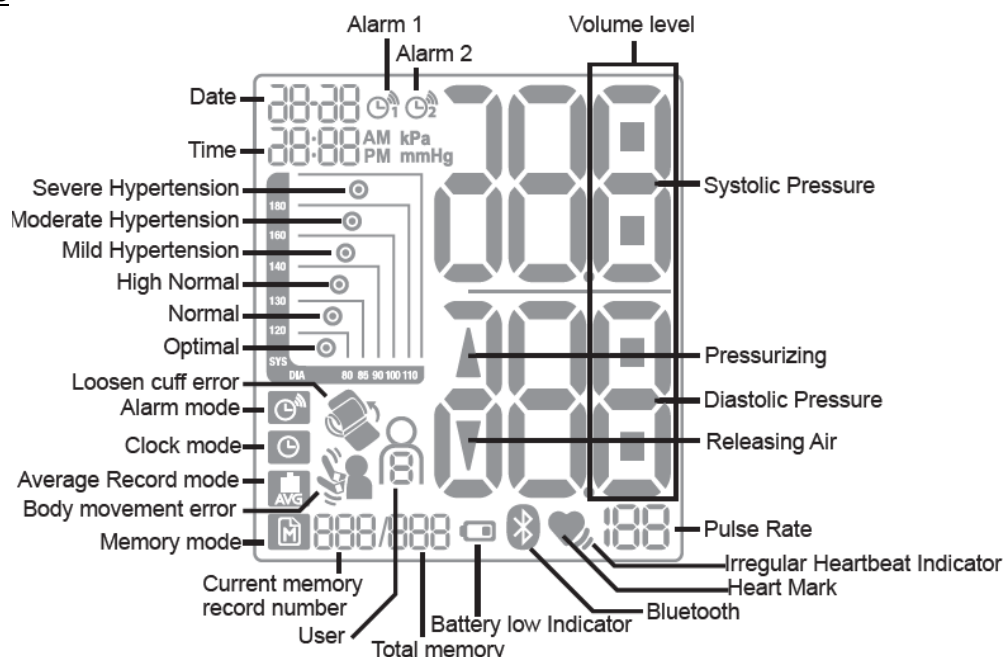

PART IDENTIFICATION

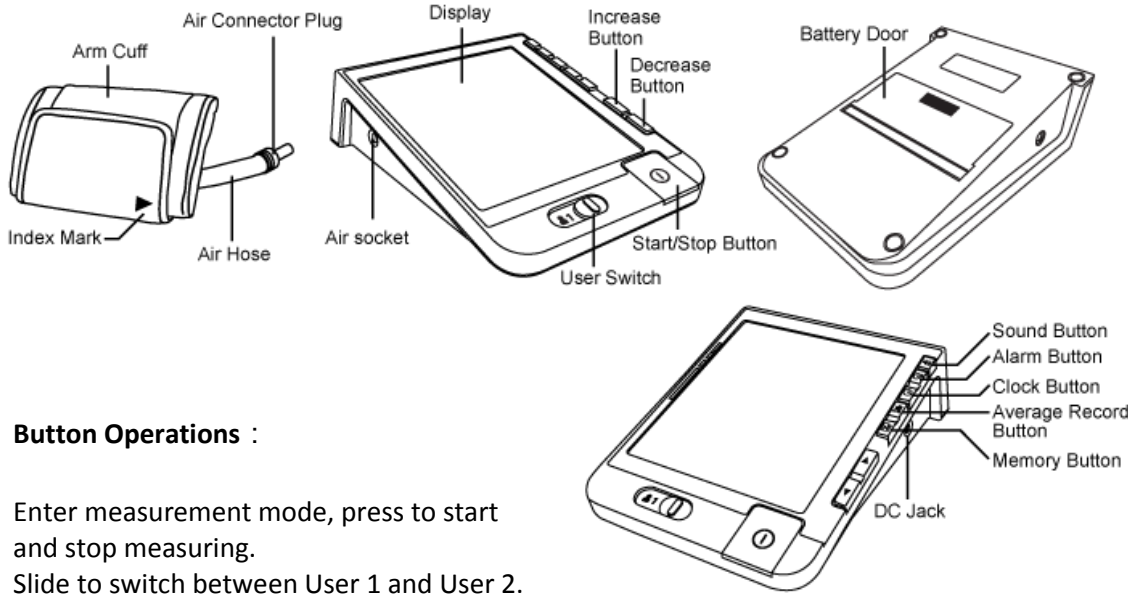
Button Operations :

- ① Enter measurement mode, press to start and stop measuring.
- 1/2 Enter measurement mode, press to start and stop measuring.
- 🔊 Enter sound mode.
- 🕒 Enter Alarm mode.
- 📅 Enter Clock mode.
- 📊 Enter Average Record mode.
- 📁 Enter Memory mode.

C. HOW TO USE IT

1) Insert or replace batteries

1. Push the battery cover located on the bottom side on the unit as shown in the figure.
2. Remove old batteries and insert new ones. Use LR6 or AA alkaline batteries and use batteries of the same type only.
3. Make sure the battery polarities (+) and (-) match the making on the battery compartment.
4. Place back the battery door.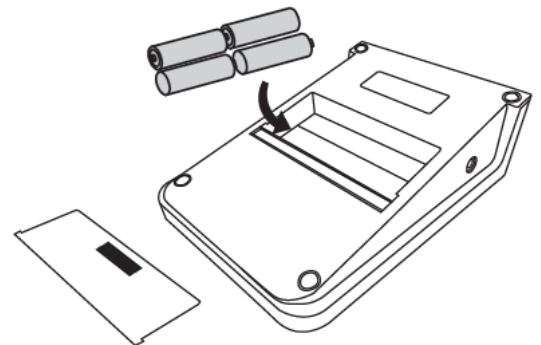


NOTE:

- Insert the batteries as shown in the battery compartment. If not, the device will not work or even be damaged.
- When battery power becomes weak, the [] icon and "E6" appear in the display, replace all batteries with new ones. Do not mix old and new batteries.
- Never use rechargeable batteries. This may damage the unit.
- Battery life may vary with ambient temperature and may be shorter at low temperature.

2) TO SET DATE AND TIME

After inserting batteries or press and hold [] button in off mode for 5 sec to enter clock mode

1. "Year" will blink on display automatically
2. Press [▲]/[▼] button to select year
3. Press [⏻] button to confirm and "Month" will blink
4. Press [▲]/[▼] button to select month
5. Press [⏻] button to confirm and "Day" will blink
6. Press [▲]/[▼] button to select day
7. Press [⏻] button to confirm and "Hour" will blink
8. Press [▲]/[▼] button to adjust to desired hour
9. Press [⏻] button to confirm and "Minute" will blink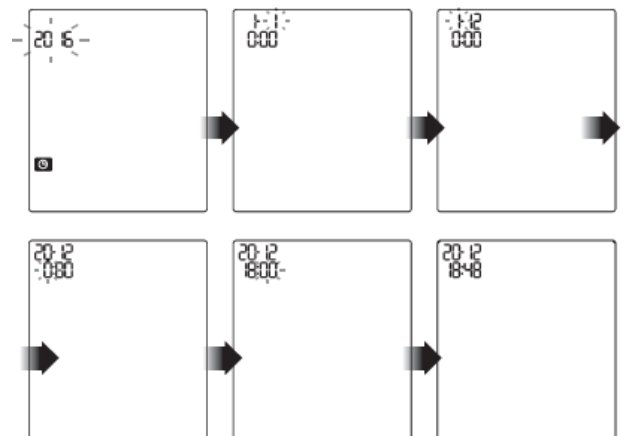


10. Press [▲]/[▼] button to adjust to desired minute
11. Press [⏻] button to confirm and settings are done

3) TO SET VOLUME

To set volume in measurement mode, press [⏻] button to turn off the unit. Press [🔊] button to enter sound mode. To set volume in other modes, press [] button to enter sound mode directly.

Press [▲]/[▼] to turn up or down the volume.
Press [⏻] button to confirm and exit.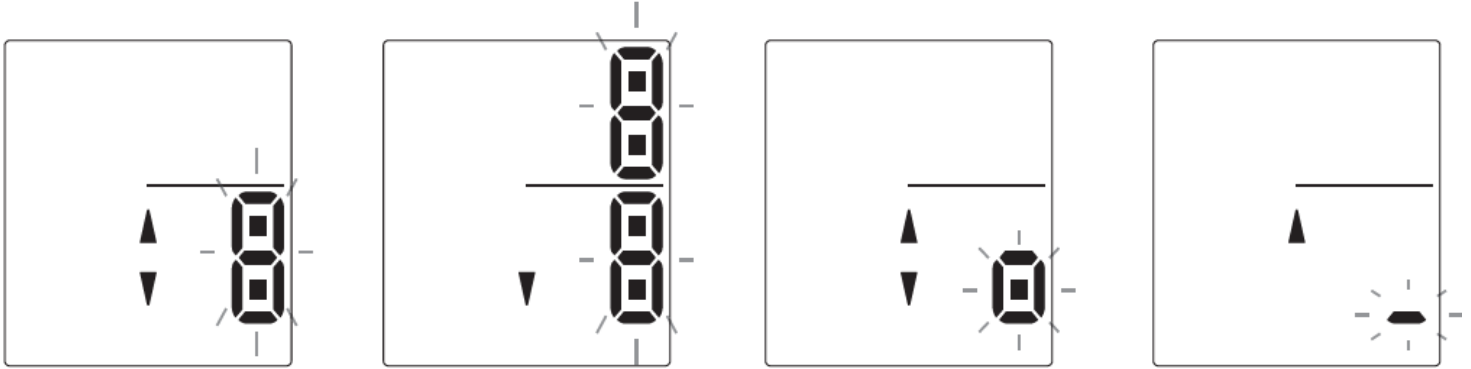

4) TO SET ALARM CLOCK

To set alarm in measurement mode, press [⏻] button to turn off the unit.

Press [🔔] button to enter alarm mode. The alarm icon is flashing on the display.

Press [🔔] button again to select between [🔔₁] and [🔔₂].


When the desired alarm icon is flashing, press [▲]/[▼] button to switch between ON [🔔₁] and OFF [🔔₁].

To confirm your selection, press and hold [🔔] button until the alarm icon stop flashing.

3. When alarm is ON, "Hour" will blink. Press [▲]/[▼] button to adjust the desired hour. Press [⏻] button to confirm.

4. "Minute" will blink. Press [▲]/[▼] button to adjust to desired minute. Press [⏻] button to confirm and settings are done. Press [⏻] button to exit the alarm mode and turn the unit off.


5. To turn off the alarm, enter the alarm mode and select the desired alarm icon. Press [▲]/[▼] button to switch to the alarm OFF [🔔₁] icon. The alarm is turned off. Press [⏻] button to exit and turn the unit off.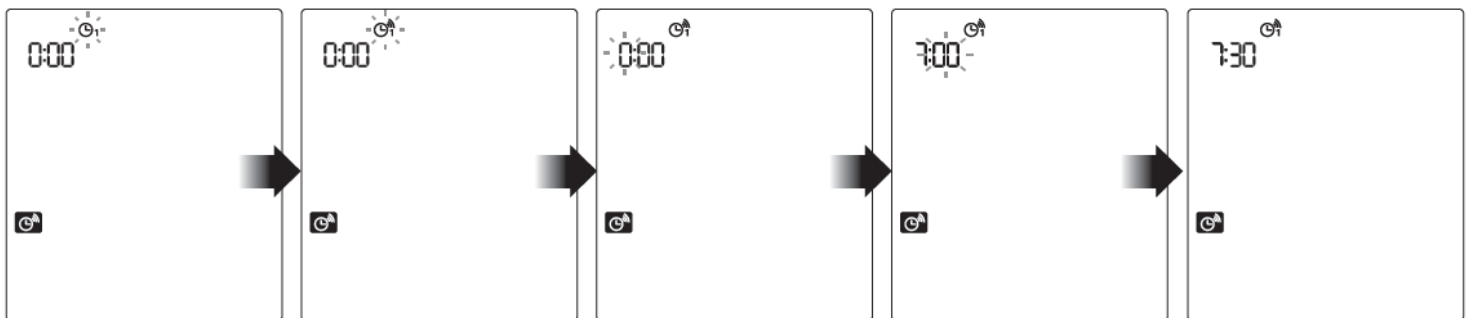

5) TO APPLY ARM CUFF

1. Pass the end of the cuff furthest from the tubing through the metal ring to form a loop. The smooth cloth should be the inside of the cuff loop.
2. Put your left arm through the cuff loop. The bottom of the cuff should be approximately 2 – 3 cm above the elbow. The white artery mark on the cuff should lie over the brachial artery on the inside of the arm. The tube should run down the centre of arm and even with the middle finger. Do not place the arm cuff over heavy clothing (e.g. jacket or sweater sleeve) as the blood pressure monitor will not be able to take proper measurement. Blood circulation in the arm should not be restricted by tight clothing or other objects.
3. Pull the cuff so that the top and bottom edges are fitted evenly around your arm.
4. When the cuff is positioned correctly, press the sewn hook material firmly against the pile side of the cuff.
5. Make sure the cuff fits snugly around your arm. The cuff should make good contact with your skin.

6. If the cuff is assembled correctly, the sewn hook material will be on the outside of the cuff loop and the metal ring will not touch your skin.

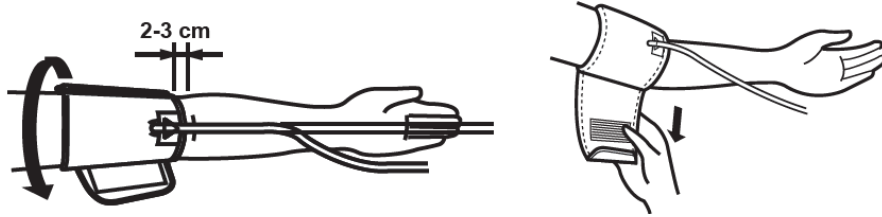7. Insert the cuff tubing into the socket on the left side of the unit. Make sure it is inserted firmly in the main unit that there are no kinks in the cuff tubing.

BEFORE MEASURING BLOOD PRESSURE

1. Applying the arm cuff, following the instruction in “TO APPLY ARM CUFF”.
2. Rest at least for 5 minutes before each measurement. Otherwise there may be erroneous results.
3. Sit comfortably on a chair with your feet flat on the floor. Do not cross your feet. Place your left arm on a table so the cuff is level with your heart.
4. Relax your arm and turn your palm upward.
5. Relax, keep still and do not talk during the measurement in order not to distort the result.

NOTE:

- This instrument is intended for use on and by adult only. Do not use this device on or by children, toddlers and infants.
- For reliable monitoring and reference of blood pressure, it is recommended to do the measurement daily at the same time.
- In order to obtain a resting condition blood pressure, do not take alcoholic and caffeinated beverages, smoke, exercise or bath for at least 30 minutes before taking a measurement.
- To minimize measurement variations due to physical activity, relax for five to ten minutes before taking a measurement. If you are excited by emotional stress or muscular movement, the measurement result may be inaccurate.
- You should not be physically tired or exhausted while taking measurement.
- Perform measurements in a quiet and relaxed environment at room temperature.
- Remain relax, still and do not speak during measurement since the accuracy of any blood pressure measurement with this device can be affected by those multitude of causes.
- Always wait at least 5 minutes between measurements to allow the blood circulation in your arm to return to normal. You may need to increase the waiting time depending on your individual physiological characteristic.
- Should the device detect an abnormal condition, it stops the measurement and will display an error code. See “TROUBLESHOOTING” for more details.
- This device measures blood pressure based on heartbeat. If you have a very weak or an irregular heartbeat, the device may have difficulty in determining your blood pressure. (An irregular heartbeat is defined as a heartbeat that varies by 25% from the average of all heartbeats during measurement.)
- Do not exert any kind of pressure on the air tube during measurement, e.g. laying your arm or any other object on the air tube. This could cause incorrect measurements.

6) TO MEASURE YOUR BLOOD PRESSURE

Press [ⓘ] button to turn on the unit and enter measurement mode.

To measure your blood pressure in other modes, press [ⓘ] button to turn off the unit and press [ⓘ] button again to enter measurement mode.


Follow the instruction in “BEFORE MEASURING BLOOD PRESSURE” to make yourself ready.

The user icon is flashing on the display, use the user slide switch to switch between user 1 [👤1] and user 2 [👤2].

When the selection is finished, the measurement will start automatically.

The cuff starts to inflate. It is normal for the cuff to feel very tight.

Once the pulse is detected, the [♥] blinks with each pulse beat, indicating that the measurement is in progress.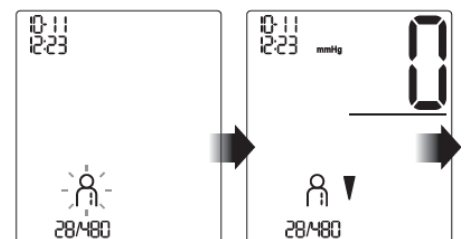


T 4780 / MD4780

When the measurement is complete, the systolic and diastolic pressure and pulse rate are displayed and stored.


NOTE:

You can stop inflation by pressing [⏹] button at any time.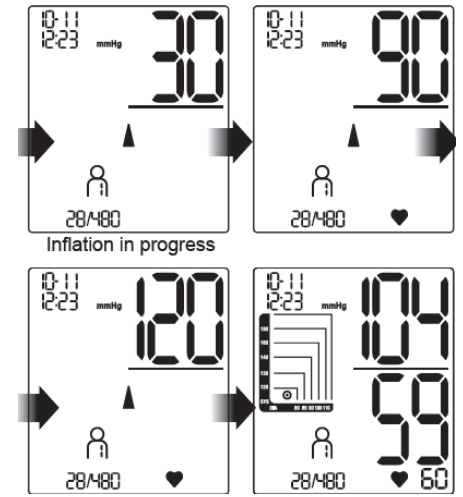

D. HOW TO READ THE RESULTS

Each of the six segments of the bar indicator corresponds to the WHO blood pressure classification.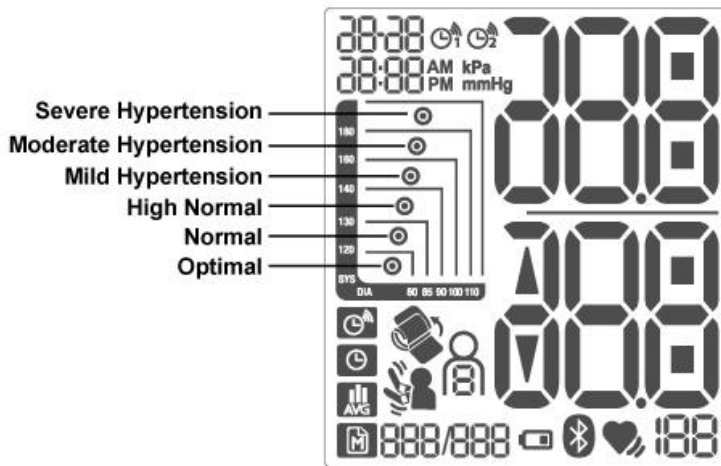

E. TO RECALL PREVIOUS MEASUREMENT DATA

To recall previous measurement data in measurement mode, press [⏹] button to turn off the unit. Press [M] button to enter memory mode.

To recall previous measurement data in other modes, press [M] button to enter memory mode directly.

1. Press [M] button to enter memory mode, the latest measurement record is displayed.
2. Select the target user ID by sliding the user switch.
3. Press [▲] or [▼] button to view previous or older measurement records.

F. TO RECALL AVERAGE MEASUREMENT DATA

To recall average measurement data in measurement mode, press [⏹] button to turn off the unit. Press [AVG] button to enter average record mode.


To recall average measurement data in other modes, press [AVG] button to enter average record mode directly.

1. Press [AVG] button to enter average record mode, the average blood pressure for the latest 3 measurements is displayed.
2. Select the target user ID by sliding the user switch.
3. Press [▲] or [▼] button to view average data for latest 7 days AM record (5:00 - 9:00 am) and average data for latest 7 days PM record (6:00 - 8:00 pm).

G. TO DELETE MEASUREMENT DATA

- To delete any single stored memory for the selected user


1. Press [M] button to enter memory mode, the latest measurement record is displayed.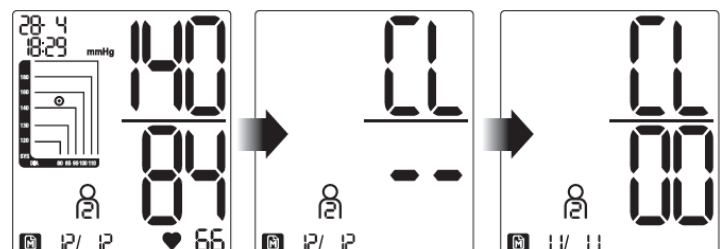

2. Select the target user ID by sliding the user switch.
3. Press [▲] or [▼] button to choose the target measurement data.
4. Press and hold [M] button for 3 seconds until the display shown "CL --".
5. Press [M] button again to confirm. The display shown "CL 00". The memory is deleted.


- To delete ALL stored memory for the selected user

1. Press [AVG] button to enter average record mode, the latest average record is displayed.
2. Select the target user ID by sliding the user switch.
3. Press and hold [AVG] button for 3 seconds until the display shown "CL --".
4. Press [AVG] button again to confirm. The display shown "CL 00". All memories for the selected user are deleted.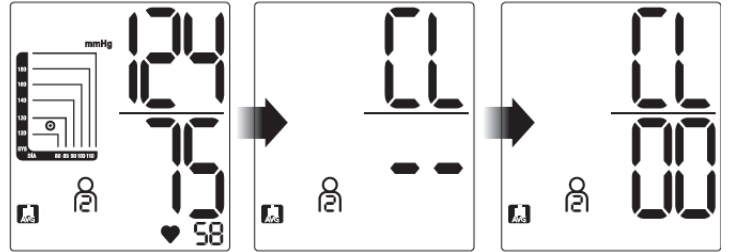


H. TIPS FOR MEASUREMENT

Measurements may be inaccurate if taken in the following circumstances.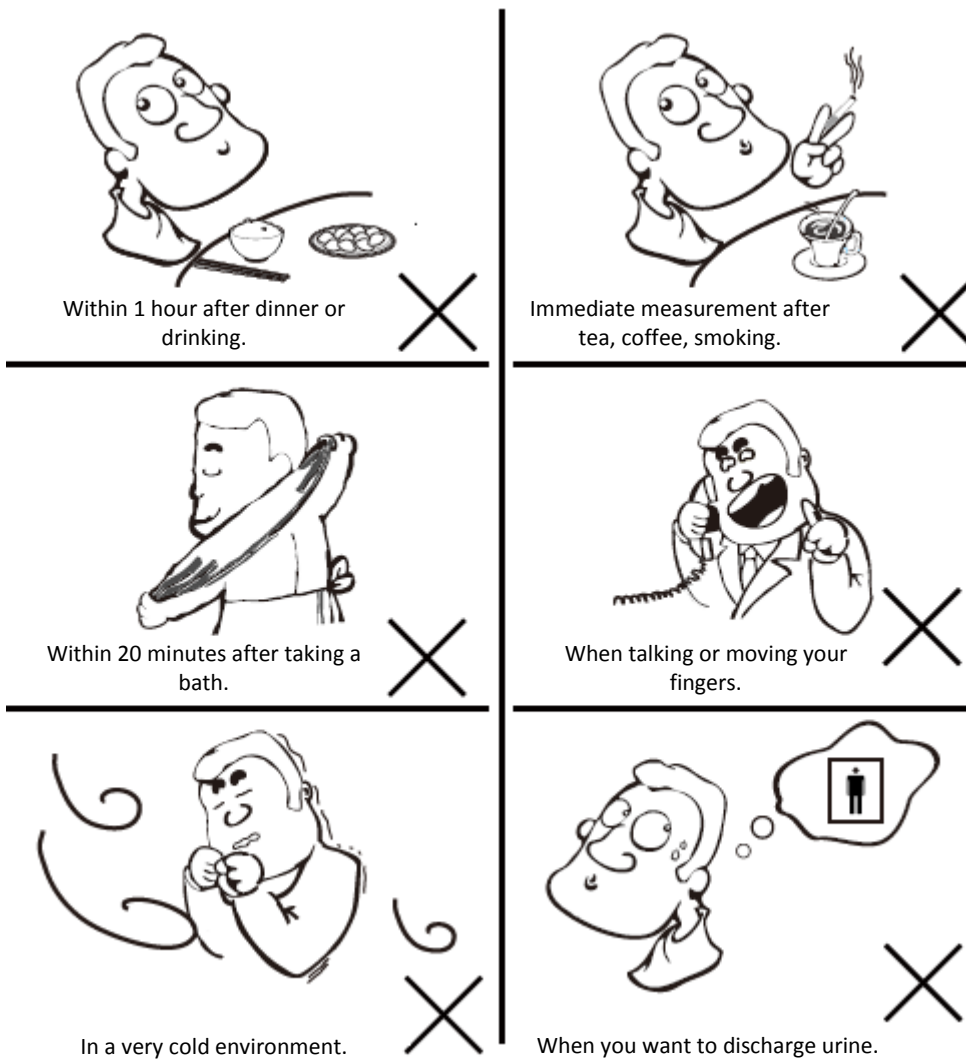

I. WHAT ARE SYSTOLIC PRESSURE AND DIASTOLIC PRESSURE?

When ventricles contract and pump blood out of the heart, blood pressure reaches its maximum value, the highest pressure in the cycle is known as systolic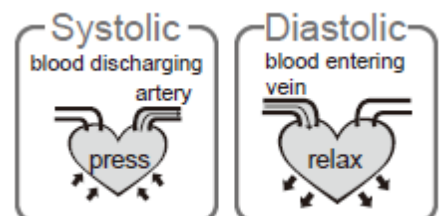


pressure. When the heart relaxes between heartbeats, the lowest blood pressure is diastolic pressure.

J. WHAT IS THE STANDARD BLOOD PRESSURE CLASSIFICATION?

The blood pressure classification published by World Health Organization (WHO) and International Society of Hypertension (ISH) in 1999 is as follows:

CAUTION: only a physician can tell your normal BP range. Please contact a physician if your measuring result falls out of the range. Please note that only a physician can tell whether your blood pressure value reached a dangerous point.

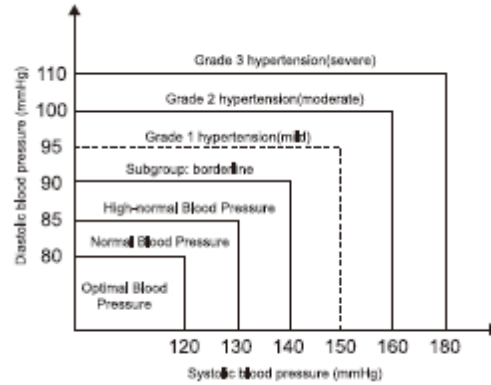K. IRREGULAR HEARTBEAT DETECTOR

An irregular heartbeat (IHB) is detected when a heartbeat rhythm varies while the unit is measuring the systolic and diastolic blood pressure. During each measurement, the monitor records the heartbeat intervals and calculates the average. If any average is larger than or equal to 25%, the irregular heartbeat symbol appears on the display when the measurement results appear.

CAUTION: The appearance of the IHB icon indicates that a pulse irregularity consistent with an irregular heartbeat was detected during measurement. Usually this is NOT a cause for concern. However, if the symbol appears often, we recommend you seek medical advice. Please note that the device does not replace a cardiac examination, but serves to detect pulse irregularities at an early stage.

L. TROUBLE SHOOTING

Nothing appears in the display, even when the power is turned on	Batteries are drained	Replace all batteries with new ones
	Battery polarities are not in the correct position	Re-install the batteries with their negative and positive ends matching their indicated in the battery compartment
	Loose in plug or contact with outlet (IF AC adaptor is used)	Check the wiring to make sure plug & outlet are properly secured
ERROR code 1 (E1) appears	The cuff position is not correct	Sit comfortably and still. Ensure that the cuff is at the same level as the heart
ERROR code 2 (E2) appears	You moved your arm or body during measurement	Make sure you remain very still and quiet during the measurement
ERROR code 3 (E3) appears	The cuff position is not fastened properly	Fasten the cuff correctly

ERROR code 3 (E3) appears	The cuff may not be applied	Check whether tube connection of the cuff is secured to the unit properly
ERROR code 4 (E4) appears	The unit does not measure	If you have a very weak or irregular heart beat, the device may have difficulty in determining your blood pressure
	There is a measuring Error	Sit comfortably and still. Fasten the cuff again carefully
ERROR code 5 (E5) appears	Cuff over inflated	The measurement range is over 300 mmHg. It is recommended to see the doctor as soon as possible.
ERROR code 6 (E6) appears	Low battery	The battery power is too low to function. Replace the batteries with new ones.
The monitor keeps inflating	Circuit locked	Remove and reinsert the batteries and then proceed to take measurement again.

M. CARE AND MAINTENANCE

Put in a dry place and avoid the sunshine.

Avoid touching water.

Avoid intense shaking and collisions.

Avoid dusty and unstable temperature environment.

Avoid washing the cuff. Clean the product only with a dry or slightly damp, lint-free cloth.

Do not use abrasive cleaners to clean the unit.

N. SPECIFICATIONS

Display mode : digital LCD

Measurement range :

- Systolic measurement pressure 60mmHg~250mmHg (8.0kPa~30.7kPa)
- Diastolic 30mmHg~200mmHg (5.3kPa~17.3kPa)
- Pulse value 40-180 beat/minute

Accuracy :

- Pressure 5°C-40°C within +/-3mmHg (0.4kPa)
- Pulse value +/-5%

Powered by 4 x AAA alkaline batteries

Normal working condition :

- A temperature range of +5°C to +40°C
- A relative humidity range of 15 to 93%, non-condensing, but not requiring a water vapour partial pressure greater than 50 hPa
- An atmospheric pressure range of 700 hPa to 1060 hPa.

Measurement preimeter of the wrist: about 22cm - 36cm

The European directive 2012/19/EC about electronic and electric waste, requires that you can't throw away defective domestic appliance with common waste. Used device has to be collect separately to optimize the recuperation rate and the recycling of the materials to reduce the impact on the health and the environment.

This appliance is not intended for use by persons (including children) with reduced physical, sensory or mental capacities, or by persons lacking in experience or knowledge, unless they are supervised or instructed on the use of this apparatus by a person responsible for their safety. Children should be supervised to ensure they do not play with this appliance.

Precision and precautions about the battery use

- Throwing away a used battery in the environment or with household waste pollutes and avoids the recuperation of reusable materials.
- It is important to limit the battery consumption and to respect the following rules:
 - Favor alkaline battery (last longer than saline battery) and when it's possible use rechargeable battery.
 - Leave used batteries and accumulators in adapted containers that you can find in some retailers. In that way, metals don't pollute the environment and the health.
- Battery has to be inserted by respecting the polarity show on the apparel and the battery. An incorrect positioning can damage the apparel, cause leaking and even provoke fire and the explosion of the battery.
- To assure a good functioning, battery has to be in a good condition. If there is a problem, put new battery.
- Never try to recharge battery not made for this. It could leak, warm up, cause fire or explode.
- Replace all the battery at the same time. Never mix up alkaline battery with saline battery.
- Used battery has to be removed.
- Also, remove the battery if you're not using the apparel for a long time.
- Never try to short-circuit the terminal of a battery.
- Never throw away the battery in fire because they can explode.
- Take out the accumulator from the apparel before reload it.
- We recommend that an adult watch if a child change the battery to respect those rules.
- If you swallow a battery, go immediately to your doctor or the hospital.
- For the best ongoing performance, we recommend using only fresh, high-quality, alkaline batteries.

Importé par / Importiert von / Geïmporteerd door / Imported by :
CAPTELEC, 59170 Croix - FRANCE

Carton et emballage papier à
trier / Kartons und
papierverpackung recyceln /
Verpakkingsmateriaal in karton
en papier te sorteren / Recycle
carton and paper packaging