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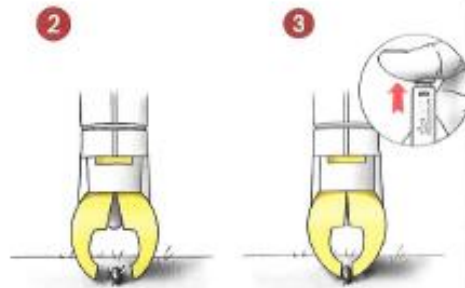
Operating Instructions

Tick Removal

☛ Choose the right gripper head. For large ticks, the closed, for small ones the open yellow gripper head is recommended.

- 1 Press the button with your thumb. The gripper head turns. Continue to press the button completely down to fully open the gripper head.
- 2 Carefully position the opened gripper head exactly vertically to the skin surface so that the tick is centered in the gripper tips.
- 3 Pay attention to the tick and release the button very slowly. The gripper head closes.
- 4 Let the button slowly slide all the way back. The gripper head twists out the tick.
- 5 The tick is inside the gripper head. For very small ticks occasionally several trials may be required.

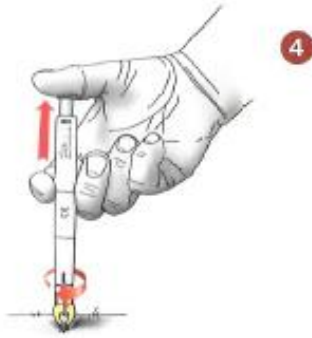
Exchange of the gripper head

Depending on the tick size, choose the right gripper head. It is also advised to replace the gripper head in case of contamination for hygienic reasons.

- 6 To replace the gripper head slowly pull it off the clasp which is located between the shaft of the Tickgripper and the gripper head. There will be a slight resistance which is normal.
- 7 To attach the new gripper head hold the Tickgripper shaft and gripper head aligned to each other at the given positioning marks. Click the gripper head onto the clasp located on the Tickgripper shaft.

Important: Do not tilt, twist or jam! Remove or affix the gripper head only along the axis of the Tickgripper shaft.

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 **tickSAFE**[®] **MED**
www.ticksafe.com

internal mechanical parts. The Tickgripper is not a toy and thus should not be handled by small children.

4 Store the Tickgripper carefully to avoid squeezing the gripper head. Deformation of the gripper head can damage its precise closure, which is essential for the removal of small ticks. Be sure to replace the protective cap over the gripper head after use.

5 Large ticks can burst during removal and contaminate the Tickgripper with blood. For hygienic reasons, a change of the gripper head is recommended in this case.

6 The Tickgripper does not require any special care. If the sensitive precision mechanics "jerk", dip the Tickgripper after removal of the gripper head shortly in a solution with dishwashing detergent (5 parts water, 1 part detergent) and press the push button until the mechanism runs free again.

By following these instructions, the Tickgripper will become a perfect aid for tick removal.

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tickSAFE GmbH