Actimo Premium

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Actimo Premium

UK

Carefully read the complete instruction manual before using Actimo Premium for the first time. Keep the instruction manual safely, so that it is available for later use.

Intended purpose

Rollator for supporting people in walking who are restricted in their movement and mobility. The rollator is foldable, has a seating function, a holder for walking sticks and a carrying case (optional) for the transport of utensils or purchases.

Indication

Actimo Premium can be used in case of mild balance problems and reduced mobility, due to loss of strength or reduced movement. To maintain the physical activity, Actimo Premium is suitable for use indoors and outdoors.

Instructions / Assembly

1. Remove the walker from the box and check the contents and the packaging for damage or completeness.

The rollator is assembled and supplied in folded form in a cardboard / plastic bag.

2. Place the Actimo Premium flat on the floor and unfold it by pulling the handles. The oval plastic holder and the frame tube must be inserted together completely before the locking lever is closed (figure 1).

3. For fixing, pull the locking lever from the frame on both the sides (Figure 2) and rotate it 180° until it points forward. Make sure that the handle tube is upright and that the plastic holder firmly locks into position in the frame. The washer must sit correctly at the end of the locking lever. Now turn it around 180 degrees until it is parallel to the frame and then press the lever in the appropriate slot (Figure 3). Make sure that you do not press the locking lever too tightly before the handle tubes are not sitting firmly on the frame in order to avoid damage!

Fig. 1

Fig. 2

Fig. 3

4. Then you open the clip on the side of the seat and press down the seat surface with the palm of your hand. Now the Actimo Premium is completely unfolded. Check whether the seating surface has fully fit into the plastic holders. Check on the outer sides whether the frame is securely connected to the plastic holders.

5. Adjust the handles at waist height, so that the arms are flexed by about 20 ° -30 ° while sliding. Loosen the knob screw at the tube holder, push or pull the handle tube to the desired height (holes for fixing) and tighten the knob screw again. Ensure that the handles are at the same height and do not pull the entire grip tube from the frame when adjusting it!

6. To fold the rollator together, pull the seat surface upwards. Loosen the locking screws on the frame and fold the handle bars to the back toward the ground.

Attaching the bag

1. Remove the covers (caps) of the holes at the rear sides of the frame and screw down the suspension device.

2. Attach the bag using the Velcro fasteners on the suspension device and the lower portion of the frame.

Using the brakes

1. To activate the parking brake, press the brake lever downwards. To loosen this, pull the brake lever back up. ALWAYS use both parking brakes simultaneously so that Actimo Premium remains fixed.
2. For short brake function or lowering the speed, pull the brake lever upwards simultaneously so that Actimo Premium does not rotate sideward.

Safety instructions

1. Before each use, check whether the rollator is completely unfolded and whether the handles are at the same height.

2. Test the brakes before using Actimo Premium and only sit on the seat when the parking brake is applied. Do not push the rollator after you sit down! There is danger of falling!
3. Do not sit on the seat when the back strap is not worn or the rollator is not standing level.
4. Do not push people with the rollator and do not hang any objects on the handles.
5. Check the brakes after each height adjustment and make sure that the brake cables are not pinched.
6. Do not use the rollator on stairs, but only on level ground! All four wheels should stand on the floor; otherwise there is danger of tipping!
7. Actimo Premium can tolerate a load of up to 136 kg! The carrier bag can be loaded up to a max. 5 kg and is intended solely for the transport of goods!
8. Ensure that the loaded bag affects the manoeuvrability of the rollator and do not walk backwards when using the rollator!
9. Do not use a defective rollator and contact the authorised dealer if you notice any material changes or signs of strong wear and tear.

Cleaning

You should clean/disinfect the rollator at regular intervals (mandatory at the time of user change). Use a mild soap without abrasive additives and a soft cloth. You can perform the disinfection using a commercial agent according to the manufacturer's instructions, e.g. with Descosept AF. Do not use abrasive or corrosive cleaning agents or high-pressure and steam cleaners!

Make sure that no dirt is deposited on the tires or brakes. This can be removed with a soft brush and warm water.

Maintenance

Actimo Premium is maintenance-free. If you notice a reduction in braking force, a sluggishness of the wheels or change in the nature of the material, please contact your dealer.

Do not use a defective rollator and check the braking function and the tightness of the grip tubes before use!

Contraindication

The use of premium Actimo not appropriate for persons with

- cognitive impairment, low vision
- serious disorders of balance
- people with amputated limbs, joint damage

Scope of delivery and storage

Actimo Premium

Bag (optional)

Rollator garage (optional)

Operating instructions

Protect the rollator from moisture and place it at a safe place to protect the linkage and the wheels from damage!

Technical specifications/dimensions and weight

Weight: 7.8 kg

Width: 61 cm

Depth: 70 cm

Height: 77 - 90 cm

Width between handles (centreline): 49 cm

Max. Load: 136 kg

Max. turning diameter: 93 cm

Material: Anodised aluminum (not susceptible to corrosion)

Disposal

A disused rollator is a valuable source of materials that can be supplied to the industrial cycle. Therefore, please do not throw the rollator in the household waste, but rather dispose it of at a state-approved disposal site. Your local waste collection services would be pleased to provide you with all the relevant information.

Warranty

The warranty period is 24 months and shall apply to any product defects that can verifiably be attributed to manufacturing or material defect.

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