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DE

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Toilet support rail RFM

UK

Carefully read the complete user manual before using the toilet support rail RFM for the first time. Keep the instruction manual safely, so that it is available for later use.

Intended use

The toilet support rail RFM is suitable for use in the bathroom / WC for users, who require a support due to loss of strength or restrictions in mobility or when sitting down or standing up.

Usage/Installation

1. Check the carton for damage and contents for completeness.
2. Remove the four frame parts and the eight wing screws.
3. Set up the two rail-elements and place the cross brace for the floor between them before joining them with two wing screws from the outside. Make sure the round ends point forwards.
4. Slide the cross members between the rail elements and tighten them from the outside with 4 wing screws. The ends of the cross members point upwards.
5. Tighten all the wing screws firmly.
6. Slide the toilet support rail towards the front of the WC, with the opening pointing backwards and the cross brace in front of the toilet.
7. Attach the basket to the left or right.

Safety Instructions

1. Regularly inspect the toilet support rail RFM if its standing in a stable position
2. Use toilet support rail RFM only on a firm and dry surface
3. Before EVERY use, check the firmness of the screws and tighten them if necessary!
4. Do not lean laterally on the toilet support rail!

5. Use both rail handles at the same time!
6. Do not stand on the railing and pay attention to the cross brace on the ground - risk of stumbling!
7. Do not use any broken toilet support rail RFM!

Cleaning

You should regularly clean and disinfect the toilet support rail, which is intended for reuse. Use a commercially available soap solution without scouring additives and a cloth to dry. The toilet support rail can be disinfected, for example, with Descosept AF or another commercially available disinfectant. Do not use corrosive cleaning agents or high-pressure and steam cleaners!

Scope of delivery

2 rail frames, 1 cross brace (floor), 1 cross member (centre), 8 wing screws, hanging basket, user manual

Technical data / dimensions and weight

Width (total): 72 cm

Depth (total): 50 cm

Width between the inner tubes: 58.5 cm

Width between the handles: 54 cm

Height: 66.5 cm

Depth of the handles: 34 cm

Weight: 4.4 kg

Max. load on the two handles: 75 kg

Material: steel (lacquered), EVAC (handles)

Warranty

The warranty period is 24 months and shall apply to any product defects that can verifiably be attributed to manufacturing or material defect.

 Daimlerstr. 12a
25337 Elmshorn
Tel.: +49 4121 473312
Fax: +49 4121 473323
www.rehaforum.com

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